

LUBRINAT™

Intravaginal Moisturizer

PRESENTATION

Original:

Cartridge with 1 aluminum tube with 30g of gel + 10 disposable applicators.

Free sample:

Cartridge with 1 aluminum tube with 10g of gel + 3 disposable applicators.

Cartridge with 1 aluminum tube with 15g of gel + 5 disposable applicators.

Primary package

- . Enameled aluminum tube in white, with internal varnish and sealant, capacity 10g, 15g and 30g.
- . Disposable polyethylene and polypropylene applicators, natural color, individually packed.

Secondary packaging

- . carton in 250g duplex paper, printed in 4 colors, folds, crease and glue on the sides.

INFORMATION

Lubrinat™ is an intravaginal gel-like moisturizer that naturally restores vaginal moisture, lasting up to 3 days after each application. Lubrinat™ eliminates dry epithelial cells and, when used regularly, restores natural vaginal moisture.

Advantages of using Lubrinat™:

Lubrinat™ contains no hormones.

Lubrinat™ contains the exact amount to be applied.

Lubrinat™ is an intravaginal moisturizer, which provides continuous moisture (hydration) for up to three days. This helps to eliminate dry epithelial cells and, when used regularly, reestablishes natural moisture; unlike a traditional lubricant that provides temporary lubrication only to reduce friction during sexual intercourse.

Lubrinat™ will make intercourse more comfortable by restoring vaginal moisture.

Lubrinat™ can be used with condoms.

Lubrinat™ contains no fragrance, does not contain parabens, has no flavor and is not aggressive.

What is the Intravaginal Moisturizer with Sodium Hyaluronate?

It is a non-hormonal gel formulation for the treatment of vaginal dryness, it has sodium hyaluronate, a natural molecule, present in many human tissues and responsible for the maintenance of a correct level of hydration in the tissues, being therefore a natural body reservoir to water, and is responsible for maintaining the correct levels of moisture and hydration in sensitive environments.

The intravaginal moisturizer has sodium lactate, is responsible for the maintenance of vaginal pH between 3.5 and 4.5 physiological, avoiding the proliferation of fungi and local bacteria and eliminating itching, inflammation, bacterial infections and burning due to lack of vaginal hydration.

Who can use it?

Intravaginal moisturizer with sodium hyaluronate has been specially developed for women suffering from vaginal dryness of various origins whose vaginal epithelium does not produce sufficient hydration and therefore is prone to itching, inflammation, bacterial infections and painful sexual intercourse, as well as after during labor, during menopause, treatments with chemotherapies and periods of stress.

What is the composition?

Sodium hyaluronate, propylene glycol, natrosol 250HHX, sodium benzoate, methyl paraben, sodium lactate, lactic acid, cremophor, deionized water.

How to use?

Applications are recommended every 3 days, reducing application spacing if necessary without damaging the natural flora of the vagina, or as directed by a physician.

Instructions for Use - How to apply




1- Separate an applicator and open the cap of the tube.



2 - To fit the applicator into the nozzle of the tube by tapping it slowly.



3 - After applying the applicator horizontally, gently squeeze the base of the tube with your fingers, forcing the gel into the applicator, filling it to the recommended limit.

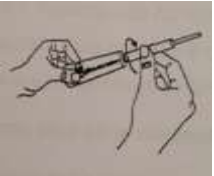

4 - Remove the applicator and close the tube.



5 - Lying on your back, with your legs bent, gently insert the applicator into the vagina as deeply as possible and apply the contents of the applicator inside the vagina. The gynecological applicator is disposable, after use it unusable.

How do I know I am using the correct amount?

The disposable applicator that comes with the Intravaginal Moisturizer has a mechanism that, when filled according to instructions, provides 3g of gel, indicated by application.

Can it be used as an intimate lubricant before intercourse?

Yes. However, if applied regularly, there will be no need for the application prior to the sexual act, since the intravaginal moisturizer is a treatment for vaginal dryness.

Can other vaginal creams be used?

Use with other vaginal creams should be directed by your doctor.

How long should it be used?

It can be used continuously or at the discretion of the physician.

What precautions to use?

- Use the Intravaginal Moisturizer under medical supervision.
- Indicated for intravaginal use only.
- The product can be used during menstruation.
- If local irritation occurs, discontinue use and seek medical advice.
- Do not use after expiration date.
- The applicators are disposable, after the use unusable them.
- Do not use the product if the packaging has been tampered with.
- Keep out of the reach of children
- Store and store the product at room temperature (15 ° C to 30 ° C), protected from light, heat and moisture.

What are the unwanted effects of Intravaginal Moisturizer?

To date there are no reports of unwanted effects with the use of the Intravaginal Moisturizer. This product is contraindicated in case of hypersensitivity to any of the components of the formulation.

How should it be stored?

Store at room temperature (between 15°C and 30°C) and protected from light, heat and moisture.

LOT NUMBER, MANUFACTURER'S DATE AND VALIDITY PERIOD: VIDE CARTRIDGE.

ANVISA Registration under N^o: 8.02.886-1 (KH96Y18L57HY)

Registration Anvisa by

Master Industry Pharmaceutical Ltda.

Rua Geraldo Andrade, nº 50 - Sebastião Dadu Arruda - Recreio - MG - CEP: 36.740-000

CNPJ: 07.279.032/0001-17

Farmaceutical Responsible: Ricardo Lawall de Andrade Cruzeiro - CRF/MG: 12.173

www.masterfarmaceutica.com.br

Brazilian industry

Manufactured by:

Helianto Pharmaceutical Ltda

Av. José Abbas Casseb, 135 - Dist. Ind. Ulisses Guimarães - CEP. 15092-609

São José do Rio Preto / SP - CNPJ. 04.506.487 / 0001-30

Farmaceutical Responsible: Maryster Ferrari Picollin – SP – CRF: 27.877

Brazilian industry

Distributed by:

Goro HealthCare FZE – License No. 5010407

Rak Free Trade Zone

City: Ras Al-Khaimah – United Arab Emirates – P.O. Box: 3965